

SUCTION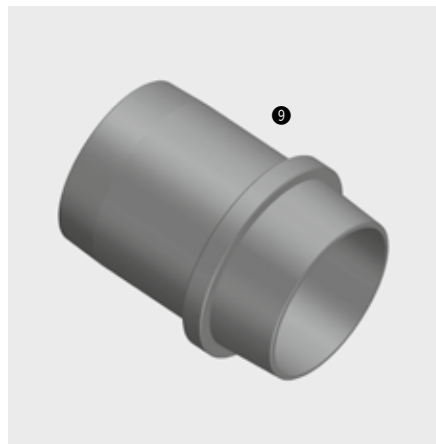

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Modo de empleo

Deutsch	04 - 08
English	09 - 13
Français	14 - 18
Italiano	19 - 23
Español	24 - 28

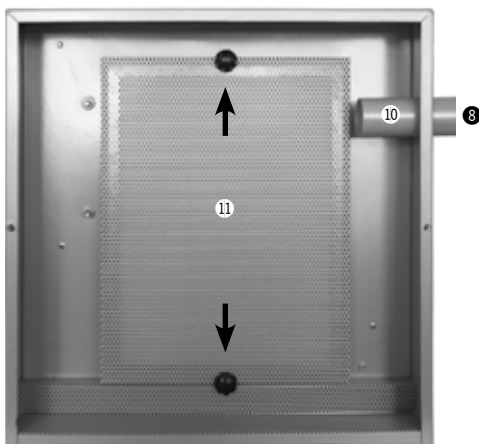ABSAUGUNG

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SUCTION

- ① On/Off switch
- ② Adjustable extraction control
- ③ Re-set button (safety switch)
- ④ Handle
- ⑤ Flap for changing the filter bag/
Micro filter
- ⑥ Socket for control cable (for automatic
operation in the milling machine)
- ⑦ Mains cable socket
- ⑧ Point of connection for extraction tube
- ⑨ Suction adapter (Art. No. 179601) for connec-
tion to the required unit (not included in the
delivery contents of the extraction unit)
- ⑩ Extraction support for filter bag
- ⑪ Micro filter
- ⑫ Flap for changing the carbon brushes

PRODUCT DESCRIPTION

The extraction is suitable for use with a dental milling machine.

The separate mains cable attachment allows automatic operation with the milling machine. The extraction unit has a special filter bag and a HEPA micro filter, which filters up to 99.97% of fine dust particles (i.e. zirconium oxide dust particles) and is classified in the fine dust particle category M.

INITIAL OPERATION

After unpacking the unit, please check for possible transport damage. Should you find any damage, please claim the damage from your supplier immediately. The extraction unit is available in 2 voltage strengths (100-120V and 230V). Before connecting to a power source, please check whether the voltage on the type plate at the back of the housing corresponds to your voltage, and make sure the ON/OFF switch is OFF.

TUBE CONNECTION

Connect the extraction tube at the designated connection point ⑦ directly to the extraction unit and the conical side to the chosen device by means of the supplied adapter ⑨.

ELECTRICAL CONNECTION

Manual Operation

Connect the extraction ⑦ to the power source using the enclosed mains cable. The extraction unit is put into operation using the ON/OFF switch ① at the positions ON/OFF.

Half-automatic operation

Connect the extraction **7** to the chosen device by connecting the mains cable to the connection cable (included with the machine to be connected). Turn the ON/OFF switch onto position ON. The extraction unit is operated once the device, to which the extraction unit is connected, is switched on.

Automatic operating (e.g. milling machine)

Connect the extraction **7** to the power source with the enclosed mains cable. Plug the control cable (e.g. from the milling machine) into the control cable socket **6** in the extraction unit. Switch the ON/OFF switch to AUTO.

MAINTENANCE

The extraction unit comprises a filter bag and a HEPA micro filter. NEITHER filter is washable! The extraction unit must never be used without both filters being present. Furthermore, only original filters may be used, otherwise the warranty claim will become invalid, and sufficient extraction of fine dust particles (according to dust particle category M) cannot be guaranteed.

Please ensure of sufficient respiratory protection. To change the filters, loosen both black screws on the right hand side of the extraction unit and remove the flap **5**.

1. To change the filter bag

After opening the flap, simply remove the filter bag from the extraction supports **10** and place a new filter bag (item no. 178610) on top.

2. To change the HEPA micro filter

The micro filter **11** is located under the filter bag. To change, loosen both black screws, remove the lattice attachment and exchange the micro filter (item no. 178611). It is recommended that the HEPA microfilter be changed at the latest after 500 operating hours.

3. Replacing the carbon brushes

(Only relevant with units, which do not have a long life motor installed - units up to the E series)

It is recommended that the carbon brushes (Art. No. 178614) be changed after 500 operating hours. The brushes should only be replaced by authorised personnel (electrician or technical service).

If the carbon brushes are used up and have not been replaced, the motor stops for reasons of safety. Once the carbon brushes have been replaced, the extraction unit is operational for the next 500 operating hours.

It is essential to remove the mains plug before changing the carbon brushes!!!

- a) Remove all hoses and cables from the extraction unit.
- b) Loosen the screws on the left side of the extraction unit **12** using a screwdriver, remove the flap and foam cover.
- c) There is one carbon brush each at the right and left side of the turbine that are retained with 2 screws. Loosen the screws, lift the mounting bracket, lift the back of the carbon brush and pull out towards the back.

- d)** Insert the replacement carbon brush upright with the guide pin facing downwards into the mounting bracket until the guide pin is fixed in position.
- e)** Tighten the screws, place on the foam cover, screw down the flap tightly, connect the mains plug.
- f)** Run the extraction unit for 20 min. on the lowest level without connecting it to a unit (e.g. extraction unit). This is to adapt the carbon brushes.
- g)** Connect the extraction unit to the unit with which it is intended to operate.

CAUTION!

As with all electronic devices, when dealing with this extraction equipment, it is always essential to handle with care and attention. The unit must be serviced and maintained regularly, in order to guarantee smooth, trouble-free operation. If necessary maintenance work is not undertaken or worn parts are not replaced, it could jeopardise the safety of the unit. In this case, the manufacturer assumes no liability whatsoever. The extraction unit is designed so that it can be connected to a power source continuously.

4. Replacing the motor

If the motor fails, after approx. 2,500 operating hours, it must be replaced. All units from the F series onwards have a long life motor, with which the carbon brushes no longer have to be exchanged. The long life motor is also available as a replacement motor for units up to the E series, which until now have not been fitted with a long life motor.

The long life motor can be identified by the "long life" symbol on the unit. The type of motor which is in the unit can be set in the milling machine software. Please contact the helpdesk for information regarding this.

The motor should only be replaced by an electrician or technical service. Please contact your technical service.

Technical Data:

D/W/H:	406 x 280 x 423 mm
Weight:	16 kg
Electrical	230 V/3,6 A
connection values:	110-120 V/7,0 A
Hertz:	230V/50 Hz
	110-120 V 60 Hz
Performance:	1000 Watt
Noise:	52 dbA
Bag capacity:	10 l
Hose -	38 mm/1,8 m
diameter/Length:	
Adapter diameter:	37 - 38 mm
	HEPA micro filter (99,97 %), Filter class H12, dust class M

Scope of delivery:

Art.-Nr.	178600-IDC: 230 V
	178600V100-V120-IDC: 100-120 V
	1 Filter bag, 1 microfilter, 1 suction hose
	1.8 m, 1 operating instructions

Accessories/Components:

Art.Nr.:	
178601	Extractor adapter
178610	Filter bag (5 items)
178611	Micro filter (1 item)
178613	Extraction hose
178614	Replacement carbon brushes with fuse (2 brushes)
178616	Long life motor 230V (from the F series onwards)
178616V100-V120	
	Long life motor 100V-120V (from the F series onwards)

!	WARNING AND SAFETY INSTRUCTIONS:	
	<p>The extraction unit was developed to be used in the dental laboratory. Initial and further operation are to be in line with the operating instructions. The manufacturer does not take responsibility for any damages that result from any other application or inappropriate handling. In addition, the user is obliged to use the device on his own responsibility and to see before use whether the device is appropriate for the intended purpose unless this is defined in the operating instructions.</p> <p>_ Please do not vacuum sharp objects!</p> <p>_ Please keep off "extraction" always, before your take out the mains plug. Otherwise extraction may be damaged - in the event</p>	<p>of which the manufacturer does not accept any liability.</p> <ul style="list-style-type: none"> _ Do not operate the extraction in a small closed room and always keep enough distance to the wall. _ The ventilator on the back has to be kept clear at all times. _ Keep the device clean. _ Exchange filter bag and micro filter. _ When changing the filter observe sufficient breathing protection and take the mains plug out beforehand! _ Keep clear the hoses. _ Protect device against humidity always. _ No contact with water or other liquidity! _ Please contact technical services in the event of motor or any other defects. _ Always connect to grounded circuits!

Issue	Possible Cause	Solution
Extraction does not start	<ul style="list-style-type: none"> _ Mains plug not inserted _ ON/OFF not switched ON _ Fuse jumped out _ Motor burnt out 	<ul style="list-style-type: none"> _ Examine mains plug _ Check ON/OFF switch _ Press reset button ③ at device _ Call technical service
Extraction extremely loud	<ul style="list-style-type: none"> _ Turbine has contact with case 	<ul style="list-style-type: none"> _ Call technical service
Insufficient extraction performance	<ul style="list-style-type: none"> _ Obstacle in system _ Filter full 	<ul style="list-style-type: none"> _ Remove obstacle _ Exchange filter
Motor switches off	<ul style="list-style-type: none"> _ Power consumption too high _ Carbon brushes are used _ Motor burnt out 	<ul style="list-style-type: none"> _ Check if filter is placed properly, press reset button _ Call technical service _ Call technical service
Air flow extremely strong	<ul style="list-style-type: none"> _ Filter not positioned correctly 	<ul style="list-style-type: none"> _ Examine filter

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Quatro Air Techn. Inc.
Amann Girrbach AG