ST.330.031

Português  Broca Countersink

Español  Fresa Countersink

English  Countersink Drill

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| Simbologia Symbology Simbología Simbologia Symbolik Symboles | Descrição / Description / Descripción / Descrizione / Beschreibung / Description |
|--|--|
|  | Tamanho do produto / Product size / Tamaño del producto / Dimensioni del prodotto / Produktmaße / Taille du produit |
|  | Código do produto / Product code / Código del producto / Codice del prodotto / Produktcode / Code du produit |
|  | Número do lote / Batch Number / Número de Partida / Numero di lotto / Chargennummer / Numéro de lot |
|  | Data de fabricação / Date of manufacture / Fecha de fabricación / Data di fabbricazione / Herstellungsdatum / Date de fabrication |
|  | Prazo de validade / Shelf life / Plazo de validad / Durata di conservazione / Mindesthaltbarkeit / Date de péremption |
|  | Quantidade / Quantity / Cantidad / Quantità / Qualität / Quantité |
|  | Material de fabricação do produto / Material used / Material de fabricación del producto / Materiale di fabbricazione del prodotto / Herstellungsmaterial des Produkts / Matériau de base du produit |
|  | Não estéril / Non-sterile / No estéril / Non sterile / Nicht steril / Non stérile |

| Simbologia Symbology Simbología Simbologia Symbolik Symboles | Descrição / Description / Descripción / Descrizione / Beschreibung / Description |
|---|--|
|  | Consulte as instruções de utilização / Refer to instructions for use / Consulte las instrucciones de utilización / Consultare le istruzioni per l'uso / Siehe Bedienungsanleitung / Consulter les instructions d'utilisation |
|  | Limite superior de temperatura / Upper limit of temperature / Limite superior de temperatura / Limite di temperatura massima / Maximaltemperatur / Limite de température supérieure |
|  | Conservar seco / Keep dry / Consérvelo seco / Conservare in luogo asciutto / Trodden lagern / Garder au sec |
|  | Manter afastado da luz solar / Keep protected from sunlight / Manténgalo lejos de la luz solar / Tenere al riparo dalla luce solare / Vor Sonneneinstrahlung schützen / Conserver à l'abri de la lumière solaire |
|  | Não utilizar se a embalagem estiver danificada / Do not use if package is damaged / No lo utilice si el envoltorio está dañado / Non utilizzare se la confezione originale è danneggiata / Bei beschädigter Verpackung Produkt nicht verwenden / Ne pas utiliser si l'emballage a été abîmé |

| Simbologia Symbology Simbología Simbologia Symbolik Symboles | Descrição / Description / Descripción / Descrizione / Beschreibung / Description |
|---|--|
|  | Marcação CE para comercialização na Comunidade Europeia / CE Mark for European Community market / Marca CE para comercialización en la Comunidad Europea / Marchio CE per la commercializzazione in Europa / CE Kennzeichen zum Vertrieb in der Europäischen Gemeinschaft / Marquage CE pour une commercialisation en Europe |
|  | Notificação exigida pelo FDA para comercialização nos Estados Unidos / Notification required by FDA for United States market / Notificación exigida por el FDA para comercialización en los Estados Unidos / Notifica necessaria per la commercializzazione negli Stati Uniti / Vom FDA geforderte Kennzeichnung für den Vertrieb in den USA / Notification exigée par FDA pour une commercialisation aux États-Unis |
|  | Representante na Comunidade Europeia / Representative in the European Community / Representante en la Comunidad Europea / Rappresentante nella Comunità europea / Vertreter in der Europäischen Gemeinschaft / Représentant dans la Communauté Européenne |
|  | Fabricante / Manufacturer / Fabricante / Produttore / Hersteller / Fabricant |

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

DESCRIPTION

The Neodent Countersink Drill is manufactured in surgical stainless steel, has cylindrical geometry and is produced in accordance to standard ISO 1797-1 - Dental rotary instruments - Shank according. . This product is used as a reaming Drill at the end of the bone drilling sequence, to mould the bone crest for adaptation of the Abutment interface of external hex and internal hex Implants. One of its ends has a reduced diameter, with no cutting power (which serves as a guide during drilling), and an angled to cylindrical cutting profile. The other end has a fitting for Contra-Angle. The Drill has a laser marking to determine drilling depth according to surgical planning. The Neodent Countersink Drill receives the application of a dark color thin carbon film on the cutting surface. This film has as main purposes: • reduction of the heat generated during osteotomy • friction reduction between the Drill and the bone • resistance increase to wear • resistance increase to oxidation.

APPLICATIONS

This product is a surgical instrument used for final drilling of bone tissue in order to prepare the bone crest to seat the cervical third of the selected Implant.

CONTRAINDICATIONS

This product does not present contraindications, once they are properly used for the indicated purposes.

HANDLING

Fit Drill in Contra-Angle. Set it at 300 rpm rotation. Perform drilling. During drilling, pressure should not be excessive, and it is necessary to provide abundant irrigation, which can be manual or combined, using motor pump. To minimize surgical trauma, which is essential for the success of the Implant osseointegration, Drill sequence should be respected. Look it up in the updated Neodent Product Catalogue or on the website: www.neodent.com.br. Do not interrupt motor rotation when Drill is inside surgical cavity, as this can hamper removal or cause Drill fracture.

SANITATION

The surgical instruments should be correctly

sanitized after each use. Do not leave the product in contact with humidity more than the time necessary for the sanitation. Proceed as follows: 1st step: Fully immerse the part in enzymatic detergent (diluted according to the manufacturer). 2nd step: Wash in ultrasonic cleaner for approximately 10 to 15 minutes. 3rd step: Rinse with distilled water in abundance, until completely remove the wastes from the solution. Nylon brushes use is recommended. 4th step: Dry with a clean and dry cloth or with compressed air. 5th step: Perform a visual inspection, observing if there are failures in the cleaning process. If there are still wastes, the part should be once again immersed in detergent - 1st step - and, if necessary, the cleaning should be performed with the aid of a nylon brush. Repeat the rinsing and drying sequence. 6th step: If applicable, place the product in an appropriate case duly sanitized and dry. 7th step: Select the packing in accordance to the sterilization process. Use preferably surgical grade paper self-sealing packing with laminated film.

PRESENTATION AND STERILIZATION

This product is unitarily provided, conditioned in blister (surgical grade film and paper type) packing. It is reusable, delivered non-sterile and should, therefore, be sterilized before use. Sterilize the product on the day before or on the day of the procedure. We preferably recommend following the sterilization method by steam autoclaving and, yet, the parameters and procedures established in standard ISO 17665-1:2006 - Sterilization of health care products. Moist heat. Requirements for the development, validation and routine control of a sterilization process for medical devices. Suitable parameters: 121°C, 30 min, at 1 atm. Alternatively, can proceed according to the instruction manual of the manufacturer of the autoclave. Sterilization validity: 7 to 15 days, provided that the product is conditioned in clean, dry and protected from the sunlight environment.

PRECAUTIONS

• Do not use the product if the packing is damaged. • This product should be sterile. • The non-replacement of the Drills, as recommended by the manufacturer, can generate improper bone heating, compromising the osseointegration success. • Do not regrind. • Do not use solutions with ammonia, acids or hydrogen peroxide, because they can damage the Drill. • Do not use anticrust products, avoid using enzymatic liquid in concentrations above 10% and dry parts that still have cleaning solution wastes, because these procedures cause oxidation. • The use of steel brushes is inadvisable. We recommend the

use of nylon brushes. • The improper planning can jeopardize the performance of the implant/prosthesis set causing failures to the system, such as loss or break of the Implant, loosening or fracture of the prosthetic screws. • Better results are obtained with the use of the Neodent products sequence. • The use of instruments and/or prosthetic abutments of other manufacturers do not guarantee the perfect function of the Neodent Implants System and exempts any guarantee of the product.

ADVERSE EFFECTS

Adverse effects will only occur if the choice and the use of instrument are improper.

POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring after the surgery and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

DATE OF EXPIRATION

Written on the label.

LIFE CYCLE

This product has a useful life of up to 20 drillings for bone densities I and II and up to 30 drillings for bone densities III and IV, providing their conditions of use are respected, including adequate irrigation and rotation. Regardless of the useful life mentioned, the professional should always check the drill sharpness. Blunt drills should be discarded. Note: Bone quality classification pursuant to Lekholm and Zarb (1985).

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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