ST.330.032

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

Neodent stainless steel surgical instruments are machined instruments for dental surgeries with presentation corresponding to their indication.

DESCRIPTION

The Punch for Bone Grafting and Orthodontic Anchorage/ Punch for Orthodontic Anchorage has a cylindrical geometry, with one end in the form of a conical point 2 mm in diameter.

Punch for Bone Grafting and Orthodontic Anchorage: has a laser marking that determines the depth of the bone deformation, so that the diameter of the mark is no greater than 0.8 mm. It is made of surgical stainless steel and has a concavity for fitting together with the Hand Wrench for Bone Grafting.

Punch for Orthodontic Anchorage: made of surgical stainless steel, it has a fixed handle made of polymer that assists in handling.

INDICATIONS FOR USE

The Punch for Bone Grafting and Orthodontic Anchorage/ Punch for Orthodontic Anchorage are surgical instruments indicated to facilitate the start of drilling of the bone tissue.

APPLICATIONS

Punch for Bone Grafting and Orthodontic Anchorage: used for demarcation of the place of insertion of the Self-drilling Screw for Bone Grafting, when it is not necessary to execute drilling of the bone tissue, especially for bone types III and IV.

Punch for Orthodontic Anchorage: used for demarcation of the place of insertion of the Self-drilling Implant for Orthodontic Anchorage, when it is not necessary to execute drilling of the bone tissue, especially for bone types III and IV.

Note: Bone quality classification according to Lekholm and Zarb, 1985.

CONTRAINDICATIONS

This product does not show contraindications, provided that it be properly used for the shown purposes.

HANDLING

Punch for Bone Grafting and Orthodontic Anchorage: fit it onto the Hand Wrench for Bone Grafting.

Position the conical tip on the place where the Self-drilling Screw for Bone Grafting will be inserted and press it using the Hand Wrench until it covers the laser marking. After use, it will leave an indentation in the bone which should be used to start insertion of the Self-drilling Screw for Bone Grafting or the Self-drilling Implant for Orthodontic Anchorage.

Punch for Orthodontic Anchorage: press the conical tip on the place where the Self-drilling Implant for Orthodontic Anchorage will be inserted and press using the handle.

After use, it will leave an indentation in the bone which should be used to start the insertion of the Self-drilling Implant for Orthodontic Anchorage.

SANITIZATION

This product should be correctly cleaned after each use. In the case of the Punch for Bone Grafting and Orthodontic Anchorage, uncouple it from the Hand Wrench for Bone Grafting.

For sanitization, proceed as follows: 1st step: Immerse completely in enzymatic detergent (diluted according to the manufacturer).

2nd step: Wash in an ultrasound washer for approximately 10 to 15 minutes. 3rd step: Rinse with plenty of distilled water, until the solution residues are completely removed. It is recommended to use nylon brushes. 4th step: Dry with a clean, dry cloth or with compressed air.

5th step: Conduct a visual inspection, observing whether there are failures in the cleaning process. If there are still residues, the piece must be immersed in detergent again - 1st step - and, if necessary, the cleaning should be done with the aid of a nylon brush. Repeat the sequence of rinsing and drying. 6th step: Select the packaging according to the sterilization process.

Preferable use self-sealing surgical grade paper packaging with laminated film. ATTENTION: Do not use descaling agents, and avoid drying pieces that still contain residues of cleaning solution, because these procedures promote oxidation. The use of steel brushes is not recommended.

The drying of the parts is of utmost importance before storage and sterilization, because the accumulation of moisture on the products is harmful and may cause oxidation.

PRESENTATION AND STERILIZATION

These products are reusable and supplied non-sterile, packed individually in a package. They are provided with a silicone protection that prevents perforation of the packaging by the conical tip of the Punch.

This protection must be removed before sterilization. These products must be correctly sanitized and sterilized before use. Sterilize the products the day before or on the day of the procedure. ATTENTION: It is not recommended to autoclave these products in their original packaging. In the case of the Punch for Bone Grafting and Orthodontic Anchorage, sterilization must be done with the Punch uncoupled from the Hand Wrench for Bone Grafting.

We preferably recommend following the sterilization method by steam autoclaving and, yet, the parameters and procedures established in standard BS EN ISO 17665-1:2006 (Sterilization of health care products. Moist heat Requirements for the development, validation and routine control of a sterilization process for medical devices). Alternatively, can proceed according to the instruction manual of the manufacturer of the autoclave. Suitable parameters: 121 °C, 30 min, at 1 atm. Sterilization validity: 7 to 15 days, provided that the product is conditioned in clean, dry and protected from the sunlight environment.

PRECAUTIONS

- Be aware in cases of patients that show signs of allergy or hypersensitivity to the chemical components of the material: surgical stainless steel.
- This product must be used sterile.
- This product should not be sharpened.
- Do not use the product if the package has been violated.
- Do not use the product with the validity expired.
- Before each procedure, make sure the pieces are properly seated.
- Make sure you have all the necessary instruments for the surgery according to surgical planning.
- Before each procedure check the conditions of the **Neodent** surgical instruments, always respecting their useful life. Replace the instruments if there is damage, markings deleted, sharpening jeopardized, deformation and wear.
- Always use the **Neodent** product sequence. The use of prosthetic components and/or instruments of other manufacturers does not ensure the perfect function of the **Neodent** Implant System

and exempts any product warranty.

- It is the professional's responsibility to use the **Neodent** products according to the instructions for use, and to determine whether it suits the individual situation of each patient.

ADVERSE EFFECTS

No adverse effects will occur, provided that the indications are complied with.

POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring after the surgery and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

DATE OF EXPIRATION

Written on the label.

LIFE CYCLE

This product should be disposed of when it loses its functionality.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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