ST.330.041

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

DESCRIPTION

The Neodent Analog Handle is manufactured in stainless steel and can be reused. It has cylindrical shape. At its tip, there is a tweezers mandrel-shaped for fixing the Analogs of Prosthetic Components, which allows the grasp for polishing and wear of Prosthetic Components.

APPLICATIONS

The Neodent Analog Handle has as objective allows the grasping for polishing and grinding of prosthetic components, as Abutments, Copings and UCLAs.

CONTRAINDICATIONS

The Neodent Analog Handle has no contraindications, providing its intended purposes and indications are correctly followed.

HANDLING

Fit the Analog (component replica) in the mandrel, so that stays fixed. Place the Prosthetic Component on the Analog and fix it by its screw. Hold the Driver with your non-dominant hand, and use the another hand to perform the necessary wearing or polishing on the component.

SANITATION

The Neodent Analog Handle is reusable and should be correctly cleaned after each use. Follow these steps for cleaning: 1st step: Fully immerse the part in enzymatic detergent (diluted according to the manufacturer). 2nd step: Wash

it in an ultrasonic cleaner for approximately 10 to 15 minutes. 3rd step: Rinse with distilled water in abundance, until completely remove the wastes from the solution. Nylon brushes use is allowed. 4th step: Dry it with a clean, dry cloth or with compressed air. 5th step: Perform a visual inspection, observing if there are failures in the cleaning process. If there are still wastes, the part should be once again immersed in detergent - 1st step - and, if necessary, the cleaning should be performed with the aid of a nylon brush. Repeat the rinsing and drying sequence. 6th step: Select the packing in accordance with the sterilization process. Preferably, use surgical grade paper self-sealing packing with laminated film. ATTENTION: Do not use anticrust products, avoid drying parts still containing cleaning solution wastes because such procedures favor the oxidation. The use of steel brushes is not recommended.

PRESENTATION AND STERILIZATION

This product is supplied unitarily, it is conditioned in plastic package. It is reusable, supplied non-sterile and should be correctly cleaned after each use.

PRECAUTIONS

- Do not use the product if the package has been violated.
- Do not sterilize the product in its original package.
- Better results are achieved with the utilization of the Neodent products sequence. The utilization of instrumentations and/or prosthetic components from other manufacturers does not assure the perfect function of the Neodent Implant System and voids any product guarantee.

ADVERSE EFFECTS

This product is used for grasping prosthetic

components for polishing and grinding. Adverse effects will only occur if the instruments are chosen incorrectly, which can cause damage to the Implant being installed.

POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring after the surgery and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

DATE OF EXPIRATION

Written on the label.

LIFE CYCLE

This product should be disposed of after it loses its functionality.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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