ST.330.062

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

DESCRIPTION

The Disposable Bone Collector is a polymer-based surgical instrument. A dedicated compartment houses a disposable sieve within the Bone Collector's body. At its active end there is a suction tube. At the other end, there is a fitting for the vacuum pump suction hose. It also has a suction control valve. The Disposable Bone Collector is supplied assembled and ready to use.

APPLICATIONS

This product is indicated for collecting autogenous bone in dental surgeries. This product is indicated for suction and storage of autogenous bone particles during drilling for Implant installation and in guided bone regeneration reconstructive surgeries for bone grafts. The disposable sieve has the function of separating blood and saline from bone particles and storing them.

CONTRAINDICATIONS

This product does not present contraindications, once they are properly used for the indicated purposes.

HANDLING

Attach suction hose to Collector's end. With the valve open, place Collector tip near drill for aspiration of saline and bone while drilling. After aspiration, keep the valve closed. Take off collector's cap and remove the sieve from inside the Collector.

PRESENTATION AND STERILIZATION

This product is unitarily provided, conditioned in blister (surgical grade film and paper type) and envelope packaging. It is delivered sterile, and ready for use. It's single use. Sterilization method: ethylene oxide.

PRECAUTIONS

- This product is of single use.
- Reuse of this product may cause:
 - adverse biological effects of residual products, microorganisms and / or substances resulting from previous uses and / or reprocessing;
 - changes in physical, mechanical and chemical properties of products, macro and micro structural, that can put in risk the desired functionality. The reuse of this product does not guarantee its safety and efficacy and disclaims any warranty of products.
 - Do not use the product if the packing is damaged.
 - This product should be used immediately after opening the package, at the time of surgery. If you do not use it, discard it.
 - Better results are obtained with the use of the Neodent products sequence. The use of instruments and/or Prosthetic Abutments of other manufacturers do not guarantee the perfect function of the Neodent Implants System and exempts any guarantee of the product.
 - Improper planning can jeopardize the performance of the Implant/Prosthesis set causing failures to the system, such as loss or break of the Implant, loosening or fracture of the Prosthetic Screws.

ADVERSE EFFECTS

Adverse effects will only occur if the choice and the use of instrument are improper.

POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring after the surgery and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

DATE OF EXPIRATION

Written on the label.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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