# ST.330.075

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English  Drivers

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Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Tamanho do produto / Product size / Tamaño del producto / Dimensioni del prodotto / Produktmaße / Taille du produit
	Código do produto / Product code / Código del producto / Codice del prodotto / Produktcode / Code du produit
	Número do lote / Batch Number / Número de Partida / Numero di lotto / Chargennummer / Numéro de lot
	Data de fabricação / Date of manufacture / Fecha de fabricación / Data di fabbricazione / Herstellungsdatum / Date de fabrication
	Prazo de validade / Shelf life / Plazo de validad / Durata di conservazione / Mindesthaltbarkeit / Date de péremption
	Quantidade / Quantity / Cantidad / Quantità / Qualität / Quantité
	Material de fabricação do produto / Material used / Material de fabricación del producto / Materiale di fabbricazione del prodotto / Herstellungsmaterial des Produkts / Matériau de base du produit
	Não estéril / Non-sterile / No estéril / Non sterile / Nicht steril / Non stérile

Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Consulte as instruções de utilização / Refer to instructions for use / Consulte las instrucciones de utilización / Consultare le istruzioni per l'uso / Siehe Bedienungsanleitung / Consulter les instructions d'utilisation
	Limite superior de temperatura / Upper limit of temperature / Limite superior de temperatura / Limite di temperatura massima / Maximaltemperatur / Limite de température supérieure
	Conservar seco / Keep dry / Consérvelo seco / Conservare in luogo asciutto / Trodden lagern / Garder au sec
	Manter afastado da luz solar / Keep protected from sunlight / Manténgalo lejos de la luz solar / Tenere al riparo dalla luce solare / Vor Sonneneinstrahlung schützen / Conservar à l'abri de la lumière solaire
	Não utilizar se a embalagem estiver danificada / Do not use if package is damaged / No lo utilice si el envoltorio está dañado / Non utilizzare se la confezione originale è danneggiata / Bei beschädigter Verpackung Produkt nicht verwenden / Ne pas utiliser si l'emballage a été abimé

Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Marcação CE para comercialização na Comunidade Europeia / CE Mark for European Community market / Marca CE para comercialización en la Comunidad Europea / Marchio CE per la commercializzazione in Europa / CE Kennzeichen zum Vertrieb in der Europäischen Gemeinschaft / Marquage CE pour une commercialisation en Europe
	Notificação exigida pelo FDA para comercialização nos Estados Unidos / Notification required by FDA for United States market / Notificación exigida por el FDA para comercialización en los Estados Unidos / Notifica necessaria per la commercializzazione negli Stati Uniti / Vom FDA geforderte Kennzeichnung für den Vertrieb in den USA / Notification exigée par FDA pour une commercialisation aux États-Unis
	Representante na Comunidade Europeia / Representative in the European Community / Representante en la Comunidad Europea / Rappresentante nella Comunità europea / Vertreter in der Europäischen Gemeinschaft / Représentant dans la Communauté Européenne
	Fabricante / Manufacturer / Fabricante / Produttore / Hersteller / Fabricant

105.001; 105.005; 105.006; 105.007; 105.008; 105.009;  
105.017; 105.018; 105.030; 105.031; 105.041; 105.042;  
105.044; 105.045; 105.046; 105.048; 105.049; 105.065;  
105.071; 105.073; 105.074; 105.087; 105.088; 105.092;  
105.109; 105.112; 105.111

105.002; 105.010; 105.012; 105.013; 105.025; 105.039;  
105.040; 105.047; 105.050; 105.066; 105.067; 105.068;  
105.075; 105.079; 105.080; 105.081; 105.089; 105.091;  
105.104; 105.115; 105.116; 105.117; 105.118; 105.119;  
105.126; 105.127; 105.128

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

## DESCRIPTION

Drivers are manufactured in stainless steel. They are available in versions with a fitting for a Torque Wrench and with a fitting for Contra-Angle, which is in accordance with standard ISO 1797-1 - Dental rotary instruments - Shank, as description below:

**Connections for Contra-Angle:** 0.9 Hex Screwdriver, 1.2 Hex Screwdriver, 1.6 Hex Screwdriver, SMART/WS Implant Driver, Slot Screwdriver, CM Implant Driver\*, Facility Screwdriver, Ti 3.5\*\*, Ti 4.1 and 4.3\*\*, Ti 5.0\*\*, II Plus\*\*, CM Neoguide 3.5\*, CM Neoguide 3.75\*, CM Neoguide 4.0\*, Narrow CM Neoguide\*, Narrow Guided Surgery CM Connection\*, Guided Surgery CM Connection\*, Guided Surgery Facility Connection, Anchorage Implant Driver, Zygomatic, Narrow SMART\*\*, Regular SMART\*\* and Wide SMART\*\*. These have a fitting for Contra-Angle or for Manual Screwdriver for Contra-Angle Connections. At their active end, their geometry is appropriate for coupling to their respective Screws/Prosthetic Components/Implants/Bone Tap.

**Connections fitting for Torque Wrench:** 0.9 Hex Screwdriver, 1.2 Hex Screwdriver, Short 1.2 Hex Screwdriver, Long 1.2 Hex Screwdriver, 1.6 Hex Screwdriver, Short 1.6 Hex Screwdriver, Slot Screwdriver, Square Screwdriver, Prosthetic Abutments Driver, Short Prosthetic Abutments Driver, Long SMART/WS Implant Driver, Short SMART/WS Implant Driver, Short CM Implant Driver, Long CM Implant Driver, Facility, Long Facility, Short Ti 3.5, Long Ti 3.5, Short Ti 4.1 and 4.3, Long Ti 4.1 and 4.3, Short Ti 5.0, Long Ti 5.0, Short II Plus, Long II Plus, Narrow CM Neoguide, Neoguide Facility. These have a square fitting, O'ring for coupling to the Torque Wrench and Bone Tap Connection Facility. At their active end, their geometry is appropriate for coupling to their respective Screws/Prosthetic Components/Implants/Bone Tap.

\*It has a silicone point for capturing and carrying the Implant. \*\*It has a pliers system for capturing and carrying the Implant.

## INDICATIONS FOR USE

The surgical connections and instruments suitable for capture, transport, and installation of osseointegrable implants.

The prosthetic connections are instruments suitable for the capture, transport and installation of prosthetic components.

## APPLICATIONS

Drivers are indicated according to description:

**Torquing or removing Screws or Prosthetic Components on Implants or intermediaries:**

0.9 Hex Screwdriver Torque Connection - Contra-Angle, 1.2 Hex Screwdriver Torque Connection - Contra-Angle, 1.6 Hex Screwdriver Torque Connection - Contra-Angle, SMART/WS Implant Driver - Contra-Angle, Slot Screwdriver Torque Connection - Contra-Angle, 0.9 Hex Screwdriver Torque Connection, 1.2 Hex Screwdriver Torque Connection, Short 1.2 Hex Screwdriver Torque Connection, Long 1.2 Hex Screwdriver Torque

Connection, 1.6 Hex Screwdriver Torque Connection, Short 1.6 Hex Screwdriver Torque Connection, Slot Screwdriver Torque Connection, Square Screwdriver Torque Connection, Prosthetic Abutments Driver, Short Prosthetic Abutments Driver.

**Installation of Morse Taper Implants:** CM Implant Driver - Contra-Angle, Short CM Implant Driver - Torque Wrench, Long CM Implant Driver - Torque Wrench. At the end of the installation, make sure that one of the six spherical indications of the connection, corresponding to the faces of the indexer of the implant (Exact), is turned toward the vestibular face.

**Installation of Facility Implants:** Contra-Angle Facility Connection, Facility Connection for Torque Wrench, Long Facility Connection for Torque Wrench. At the end of the installation, make sure that one of the spherical indications of the connection is turned toward the vestibular face.

**Installation of External Hex Implants with Inner Torque:** 3.5 Ti Connection - Contra-Angle, 4.1 and 4.3 Ti Connection - Contra-Angle, 5.0 Ti Connection - Contra-Angle, 3.5 Short Ti Connection - Torque Wrench, 3.5 Long Ti Connection - Torque Wrench, 4.1 and 4.3 Short Ti Connection - Torque Wrench, 4.1 and 4.3 Long Ti Connection - Torque Wrench, 5.0 Short Ti Connection - Torque Wrench, 5.0 Long Ti Connection - Torque Wrench. At the end of the installation, make sure that the laser marking of the active end of the connection is aligned with the hexagon of the implant.

**Installation of Inner Hex Implants:** Connection II Plus - Contra-Angle, Short Connection II Plus - Torque Wrench, Long Connection II Plus - Torque Wrench. At the end of the installation, make sure that one of the six spherical indications of the connection, corresponding to the faces of the hexagon of the implant, is turned toward the vestibular face.

**Installation of Morse Taper Implants with the NeoGuide technique:** 3.5 CM Neoguide Connection - Contra-Angle, 3.75 CM Neoguide Connection - Contra-Angle, 4.0 CM Neoguide Connection - Contra-Angle, Narrow CM Neoguide Connection - Contra-Angle, Narrow CM Neoguide Connection - Torque Wrench. At the end of the installation, make sure that one of the six spherical indications of the connection, corresponding to the faces of the indexer of the implant (Exact), is turned toward the vestibular face.

**Installation of Morse Taper Implants using the Neodent Guided Surgery technique:** Neodent Straight CM Guided Surgery Connection for Contra-Angle Handpiece, Neodent CM Guided Surgery Connection for Contra-Angle Handpiece. For installation with the Torque Meter Ratchet, couple the Hex Connection for ratchet to the Neodent Guided Surgery connections for contra-angle handpiece. At the end of the installation, make sure that one of the six spherical indications of the connection, corresponding to the faces of the indexer of the implant (Exact), is turned toward the vestibular face.

**Installation of Facility Implants using the Neodent Guided Surgery technique:** Neodent Facility Guided Surgery Connection for Contra-Angle Handpiece. For installation with the Torque Meter Ratchet, couple the Facility Bone Tap Connection for Ratchet to the Neodent Facility Guided Surgery Connection for Contra-Angle Handpiece. At the end of the installation, make sure that one of the spherical indications of the connection is turned toward the vestibular face.

**Installation of Morse Taper, Zygomatic Implants or Outer Hex with Mount:** Zygomatic Connection - Contra-Angle.

**Installation of Outer Hex Implants with Mount, WS and GT:** Hex Connection for Contra-Angle Handpiece, Long SMART/WS Implant Driver - Torque Wrench, Short SMART/WS Implant Driver - Torque Wrench.

**Installation of Implants for Orthodontic Anchorage:** Anchorage Implant Driver • Contra-Angle.

**Installation of SMART Implants:** Narrow SMART Implant Driver (Purple) for HE SMART 3.5mm Implants; Regular SMART Implant Driver (Blue) for HE SMART 4.1/4.3mm and HI SMART 3.75, 4.3, 5.0mm Implants; Wide SMART Implant Driver (Bronze) for HE SMART 5.0mm Implants. For installation with Torque Wrench fit the Hex Connection over the SMART Implant Driver - Contra-Angle.

**For formation of threads in the surgical socket with the Alvim Bone Tap and Neodent Alvim Guided Surgery Bone Tap -** Long Hex Connection for Ratchet, Hex Connection for Contra-Angle Handpiece, or Short Hex Connection for Ratchet.

**For formation of threads in the surgical socket with the Facility Bone Tap and Neodent Facility Guided Surgery Bone Tap -** Facility Bone Tap Connection for Ratchet.

## CONTRAINDICATIONS

The Drivers do not present contraindications, provided their purposes or indications are adhered to.

## HANDLING

Adapt the Driver to the Torque Wrench or to the Contra-Angle, as recommended. Place the Driver on the Screw/Prosthetic Component/Implant/instrument sets, press and apply the torque. It is recommended to keep the axial Driver along the axis of the device to be installed for torque application. The torque used should adhere to the one indicated for the Screw/Prosthetic Component/Implant/instrument sets.

Maximum recommended torque for each Driver is shown below.

• Drivers for Contra-Angle:

Item Code	Connection	Maximum Torque Indicated
105.013	Slot Screwdriver	10 N.cm
105.118	0.9 Hex Screwdriver (short)	15 N.cm
105.066	0.9 Hex Screwdriver	
105.039	Anchorage Implant Driver (long)	20 N.cm
105.040	Anchorage Implant Driver (short)	

Item Code	Connection	Maximum Torque Indicated
105.010	1.2 Hex Screwdriver	32 N.cm
105.119	Short 1.2 Hex Screwdriver	
105.012	1.6 Hex Screwdriver	
105.002	SMART/WS Implant Driver	35 N.cm
105.075	CM Implant Driver	
105.104	Facility	
105.089	Ti 3.5	
105.047	Ti 4.1 and 4.3	
105.050	Ti 5.0	
105.068	II Plus	
105.067	Zygomatic	
105.123	Guided Surgery Facility	35 N.cm (for use with Contra-angle handpiece)
105.115	Narrow SMART	45 N.cm (for use with Torque Ratchet Wrench)
105.116	Regular SMART	
105.117	Wide SMART	
105.079	CM Neoguide 3.5	35 N.cm (for use with Contra-angle handpiece)
105.080	CM Neoguide 3.75	
105.081	CM Neoguide 4.0	
105.126	Narrow Guided Surgery CM	
105.127	Guided Surgery CM	
105.091	Narrow CM Neoguide	

- Drivers fitting for a Torque Wrench:

Item Code	Connection	Maximum Torque Indicated
105.007	Slot Screwdriver	10 N.cm
105.065	0.9 Hex Screwdriver	15 N.cm
105.009	Prosthetic Abutments Driver	32 N.cm
105.044	Short Prosthetic Abutments Driver	
105.005	1.2 Hex Screwdriver	
105.041	Short 1.2 Hex Screwdriver	
105.071	Long 1.2 Hex Screwdriver	
105.006	1.6 Hex Screwdriver	
105.042	Short 1.6 Hex Screwdriver	45 N.cm
105.008	Square Screwdriver	
105.009	Long Facility	
105.111	Bone Tap Connection Facility	60 N.cm
105.018	Long SMART/WS Implant Driver	
105.001	Short SMART/WS Implant Driver	
105.073	Short CM Implant Driver	
105.074	Long CM Implant Driver	
105.087	Short Ti 3.5	
105.088	Long Ti 3.5	
105.045	Short Ti 4.1 and 4.3	
105.046	Long Ti 4.1 and 4.3	
105.048	Short Ti 5.0	
105.049	Long Ti 5.0	
105.030	Short II Plus	
105.031	Long II Plus	
105.092	Narrow CM Neoguide	

NOTE: To use the connections in the Neoguide and Neodent Guided Surgery techniques, the installation connections of the implant must be fitted until reaching the Stop.

## SANITATION

This product should be correctly sanitized after each use. Do not leave the product in contact with humidity more than the time necessary for the sanitation. Proceed as follows: 1<sup>st</sup> step: Fully immerse the part in enzymatic detergent (diluted according to the manufacturer) 2<sup>nd</sup> step: Wash it in an ultrasonic cleaner for approximately 10 to 15 minutes. 3<sup>rd</sup> step: Rinse with distilled water in abundance, until completely remove the wastes from the solution. Nylon brushes use is allowed. 4<sup>th</sup> step: Dry it with a clean, dry cloth or with compressed air. 5<sup>th</sup> step: Perform a visual inspection, observing if there are failures in the cleaning process. If there are still wastes, the part should be once

again immersed in detergent - 1<sup>st</sup> step - and, if necessary, the cleaning should be performed with the aid of a nylon brush. Repeat the rinsing and drying sequence. 6<sup>th</sup> step: Select the packing in accordance with the sterilization process. Preferably, use surgical grade paper self-sealing packing with laminated film. ATTENTION: Do not use anticrust products, avoid using enzymatic liquid in concentrations above 10% and dry parts that still have cleaning solution wastes, because these procedures cause oxidation. The use of steel brushes is also inadvisable. The drying of the parts is of utmost importance before storage and sterilization, because the accumulation of moisture on the products is harmful and may cause oxidation.

## PRESENTATION AND STERILIZATION

This product is reusable and supplied non-sterile, packed individually. This product must be sanitized and sterilized before use, when used in a clinical setting. Sterilize the product the day before or on the day of the procedure. Sterilize the product on the day before or on the day of the procedure. We preferably recommend following the sterilization method by steam autoclaving and, yet, the parameters and procedures established in standard BS EN ISO 17665-1:2006 (Sterilization of health care products. Moist heat Requirements for the development, validation and routine control of a sterilization process for medical devices). Suitable parameters: 121 °C, 30 min, at 1 atm. Alternatively, can proceed according to the instruction manual of the manufacturer of the autoclave. Sterilization validity: 7 to 15 days, provided the product is conditioned in clean, dry and protected from sunlight environment.

## PRECAUTIONS

- It is recommended to keep the axial connection along the axis of the device to be installed for the application of torque.
- Inclination of the Connection during the application of torque may result in damage and/or breakage to it.
- For use of the connections for installation of implants, check the installation speed and torque for each implant in their respective Instructions for Use. Make sure the implant is correctly seated on the connection before moving it to the surgical socket.
- If there is jamming of the CM connection for ratchet inside the implant, a slight counter-torque with the Ratchet Wrench is recommended.
- If there is jamming of the TI connection, a slight counter-torque should be applied to the ratchet (counterclockwise). To do this, reverse the direction of the torque at the end of the surgical ratchet wrench. When the direction of the ratchet is reversed, the gear is released. Therefore, always perform this procedures outside the patient's mouth.
- For use of products of the Neoguide and Neodent Guided Surgery techniques, we only recommend use by qualified professionals with knowledge of Neodent's guided surgery technique.
- Surgical planning and/or inadequate prosthesis can compromise the performance of the implant/prosthesis assembly, resulting in system failure, such as loss or fracture of the implant, loosening or fracture of components and/or prosthetic screws.

- Do not use the product if the packaging is broken.
- Do not use the product with the validity expired.
- Before each procedure, make sure the pieces are properly seated.
- Ensure that the parts are not swallowed or aspirated by the patient.
- Always use the **Neodent** product sequence. The use of prosthetic components and/or instruments of other manufacturers does not ensure the perfect function of the **Neodent** Implant System and exempts any product warranty.
- It is the professional's responsibility to use the **Neodent** products according to the instructions for use.

### **ADVERSE EFFECTS**

Adverse effects will only occur if the choice and the use of instrument are improper.

### **POST-OPERATIVE PRECAUTIONS AND MAINTENANCE**

Instruct the patient as to the need of a professional medical monitoring after the surgery and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

### **STORAGE CONDITIONS**

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

### **DISPOSAL OF MATERIAL**

After use, every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

### **DATE OF EXPIRATION**

Written on the label.

### **LIFE CYCLE**

This product should be disposed of when it loses its functionality.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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