ST.330.105

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

DESCRIPTION

The O'Ring with Cylinder for Mini Ball Attachment and Equator Attachment is a Prosthetic Component having as specific function to link the removal prosthesis to the Attachment, allowing it to be removed for maintenance and cleaning. The Titanium Cylinder has a pod-shaped cylindrical geometry with a polymer O-ring conditioned in the Cylinder's inner part. On the outer part, the O'Ring with Cylinder show retention by grooves, with the purpose of retaining the acrylic resin in the removal prosthesis, promoting a better fixation. The O'Ring with Cylinder comes with a polymer protection Disk.

APPLICATIONS

The O'Ring with Cylinder is indicated as an intermediate between the prosthesis (overdenture) and the Attachment, which is accountable for the overdenture supported mucus-implant fixation. The Protection Disk's function is to limit the resin outflow during the capture of the O'Ring with Cylinder in the mouth.

CONTRAINDICATIONS

This product does not have contraindications, provided that it is used properly for the stated purposes.

HANDLING

After fastening the Attachment to the intraoral

Implant, position the Protection Disk on the Mini Ball Attachment or Equator and fit in the O'Ring with Cylinder. Use the O'Ring Positioner for Mini Ball Attachment for a perfect alignment of the Overdenture prosthesis insertion axle. Position the prosthesis and fasten the set aided by the acrylic resin.

PRESENTATION AND STERILIZATION

Is it conditioned in a blister type package (surgical grade paper and film) and supplied unitarily. It is of single use and delivered non-sterile.

PRECAUTIONS

- It is recommended capture in mouth, one component at a time.
- It is mandatory the use of Positioner only for Mini Ball Attachment
- Do not use the product if the package has been violated.
- Do not sterilize the product in its original package.
- This product is of single use.
- Reuse of this product may cause:
 - adverse biological effects of residual products, microorganisms and / or substances resulting from previous uses and / or reprocessing;
 - changes in physical, mechanical and chemical properties of products, macro and micro structural, that can put in risk the desired functionality. The reuse of this product does not guarantee its safety and efficacy and disclaims any warranty of products.
- Better results are achieved with the utilization of the Neodent products sequence. The utilization of instrumentations and/or Prosthetic Components from other manufacturers does not assure the perfect function of the Neodent Implant System and voids any product guarantee.
- The improper planning may endanger the Implant/Prosthesis set performance, resulting in

system faults, such as Implant loss or fracture, loosening or fracturing of the Prosthetic Screws.

ADVERSE EFFECTS

Adverse effects will only occur if the instrumentation selection and use are improper.

POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring after the procedure and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

DATE OF EXPIRATION

Written on the label.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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