# ST.330.128

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English  Drill Extension

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Español  Prolongador de Fresas

Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Tamanho do produto / Product size / Tamaño del producto / Dimensioni del prodotto / Produktmaße / Taille du produit
	Código do produto / Product code / Código del producto / Codice del prodotto / Produktcode / Code du produit
	Número do lote / Batch Number / Número de Partida / Numero di lotto / Chargennummer / Numéro de lot
	Data de fabricação / Date of manufacture / Fecha de fabricación / Data di fabbricazione / Herstellungsdatum / Date de fabrication
	Prazo de validade / Shelf life / Plazo de validad / Durata di conservazione / Mindesthaltbarkeit / Date de péremption
	Quantidade / Quantity / Cantidad / Quantità / Qualität / Quantité
	Material de fabricação do produto / Material used / Material de fabricación del producto / Materiale di fabbricazione del prodotto / Herstellungsmaterial des Produkts / Matériau de base du produit
	Não estéril / Non-sterile / No estéril / Non sterile / Nicht steril / Non stérile

Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Consulte as instruções de utilização / Refer to instructions for use / Consulte las instrucciones de utilización / Consultare le istruzioni per l'uso / Siehe Bedienungsanleitung / Consulter les instructions d'utilisation
	Limite superior de temperatura / Upper limit of temperature / Limite superior de temperatura / Limite di temperatura massima / Maximaltemperatur / Limite de température supérieure
	Conservar seco / Keep dry / Consérvelo seco / Conservare in luogo asciutto / Trocken lagern / Garder au sec
	Manter afastado da luz solar / Keep protected from sunlight / Manténgalo lejos de la luz solar / Tenere al riparo dalla luce solare / Vor Sonneneinstrahlung schützen / Conserver à l'abri de la lumière solaire
	Não utilizar se a embalagem estiver danificada / Do not use if package is damaged / No lo utilice si el envoltorio está dañado / Non utilizzare se la confezione originale è danneggiata / Bei beschädigter Verpackung Produkt nicht verwenden / Ne pas utiliser si l'emballage a été abîmé

Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Marcação CE para comercialização na Comunidade Europeia / CE Mark for European Community market / Marca CE para comercialización en la Comunidad Europea / Marchio CE per la commercializzazione in Europa / CE Kennzeichen zum Vertrieb in der Europäischen Gemeinschaft / Marquage CE pour une commercialisation en Europe
	Notificação exigida pelo FDA para comercialização nos Estados Unidos / Notification required by FDA for United States market / Notificación exigida por el FDA para comercialización en los Estados Unidos / Notifica necessaria per la commercializzazione negli Stati Uniti / Vom FDA geforderte Kennzeichnung für den Vertrieb in den USA / Notification exigée par FDA pour une commercialisation aux États-Unis
	Representante na Comunidade Europeia / Representative in the European Community / Representante en la Comunidad Europea / Rappresentante nella Comunità europea / Vertreter in der Europäischen Gemeinschaft / Représentant dans la Communauté Européenne
	Fabricante / Manufacturer / Fabricante / Produttore / Hersteller / Fabricant

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

## DESCRIPTION

The Neodent Drill Extension is manufactured in surgical stainless steel, it has scheduled cylindrical shape and has a non-removable lateral Screw fixed by a purple ring. One of its extremities has fit for Contra-Angle and the another one, a fit for Drills. The non-removable lateral Screw fix the Drill on the Drill Extension. The 1.2mm Hex Screwdriver must be used to fix and unleash the Drill.

## APPLICATIONS

The Neodent Drill Extension is a instrument used in dental surgery. It is indicated for extending the Drills during surgical instrumentation, whenever the adjacent interdental height is greater than 20 mm.

## CONTRAINDICATIONS

The Neodent Drill Extension has no contraindications, providing its intended purposes and indications are correctly followed.

## HANDLING

Fit the Neodent Drill Extension in the Contra-Angle. Before adapting the Drill in the Extension, make sure that the screw is in a position that allows the correct fit of the Drill. Hold the Drill, tightening slightly the screw with a 1.2 mm Hex Screwdriver. Note 1: The fixation screw is not removable, therefore there is no danger of it falling out during handling.

## SANITATION

The Neodent Drill Extension and all surgical instruments should be correctly cleaned after each use. Do not leave in contact with dampness for prolonged periods. Follow these steps for cleaning: 1st step: Fully immerse the part in enzymatic detergent (diluted according to the manufacturer) 2nd step: Wash it in an ultrasonic cleaner for approximately 10 to 15 minutes. 3rd step: Rinse with distilled water in abundance, until completely remove the wastes from the solution. Nylon brushes use is allowed. 4th step: Dry it with a clean, dry cloth or with compressed air. 5th step: Perform a visual inspection,

observing if there are failures in the cleaning process. If there are still wastes, the part should be once again immersed in detergent - 1st step - and, if necessary, the cleaning should be performed with the aid of a nylon brush. Repeat the rinsing and drying sequence. Step 6: If applicable, place the product in a suitable case duly sanitized and dried. Step 7 - Select the package according to the sterilization process. Use preferably a self-sealing package if surgical grade paper with laminated film. ATTENTION: Do not use descaling products and avoid drying parts still containing cleaning solution wastes because such procedures favor oxidation. Also, the use of steel brushes is not recommended.

## PRESENTATION AND STERILIZATION

The Neodent Drill Extension is unitarily provided, conditioned in blister (surgical grade film and paper type) packing. It is reusable, delivered non-sterile and should, therefore, be sterilized before use. Sterilize the product on the day before or on the day of the procedure. We preferably recommend following the sterilization method by steam autoclaving and, yet, the parameters and procedures established in standard BS EN ISO 17665-1:2006 (Sterilization of health care products. Moist heat. Requirements for the development, validation and routine control of a sterilization process for medical devices). Alternatively, can proceed according to the instruction manual of the manufacturer of the autoclave. Suitable parameters: 121 °C, 30 min, at 1 atm. Sterilization validity: 7 to 15 days, provided that the product is conditioned in clean, dry and protected from the sunlight environment.

## PRECAUTIONS

- Do not use the product if the package has been violated. • This product should be sterile.
- During drilling, follow the instructions for use of the selected Drill. • Do not interrupt the rotation of the motor with the Drill inside the surgical cavity, as this can hamper removal or provoke the fracture of the Drill. • Take care to irrigate the extremity of the Drill by hand, as the Drill Extension prevents the irrigation of the Contra-Angle from reaching its extremity. • The fixation Screw has a limited course. When tightening or loosening it, stop the movement as soon as resistance is encountered. Never force the screw past the end of the course, it can lock up or become stripped. • Do not use anticrust products, avoid using enzymatic liquid

in concentrations above 10% and dry parts that still have cleaning solution wastes, because these procedures cause oxidation. • The use of steel brushes is inadvisable. We recommend the use of nylon brushes. • The improper planning can jeopardize the performance of the Implant/prosthesis set causing failures to the system, such as loss or break of the Implant, loosening or fracture of the prosthetic screws. • Better results are achieved with the utilization of the Neodent products sequence. The utilization of instrumentations and/or prosthetic components from other manufacturers does not assure the perfect function of the Neodent Implant System and voids any product guarantee.

## ADVERSE EFFECTS

Adverse effects will only occur if the choice and the use of instrument are improper.

## POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring after the surgery and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

## STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40 °C and protected from direct sunlight.

## DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

## DATE OF EXPIRATION

Written on the label.

## LIFE CYCLE

Suitable on the label.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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