ST.330.168

Português 

Haste, Montador e Removedor de O'ring

Italiano 

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

DESCRIPTION

O'ring Insertion Tool: Made of stainless steel, it has a cylindrical shape, with one end for insertion of the O'Ring of the Mini Ball Attachment and the other end for insertion of the O'Ring of the Equator Attachment. While one of the ends is inserted into the Multi-use Tool, the other end performs the function of insertion of the O'Ring on the Cylinder. **O'Ring Remover:** Made of stainless steel, it has a cylindrical shape and a sharp active tip for removal of the O'Ring from the Cylinder, and on the other end, it has a polymer handle to facilitate handling. **Multi-use Tool:** Made of polymer, it has a cylindrical shape and two different ends. One of the ends serves as a support and adaptation for the O'Ring Insertion Tool and the other end serves for handling of the assembly. **ATTENTION:** All the items are provided in non-sterile form and can be reused, according to the guidelines and constant care contained in these instructions for use.

APPLICATIONS

O'Ring Insertion Tool is indicated for replacement of the O'Ring or Equator O'Ring inside of Cylinder. O'Ring Extractor Tool is indicated for the removal of the O'Ring or Equator O'Ring of Cylinder. Multi-use Tool is indicated for fixation and handling of O'Ring Insertion Tool.

CONTRAINDICATIONS

The Multi-use Tool, O'Ring Insertion/Extractor Tool do not present contraindications, provided their purposes or indications are adhered to.

HANDLING

O'Ring Extractor Tool: remove the prosthesis from the oral cavity, place the Extractor over the O'Ring and pull it until its complete removal. The O'Ring removed lose its functionality. **WARNING:** Sanitize the prosthesis as standard procedure before installing the new. **Multi-use Tool:** select the O'Ring type that will be inserted into the Cylinder, Attachment O'Ring or Equator Attachment O'Ring. Insert the O'Ring Insertion stem with the chosen end on the Multi-use Tool. **O'Ring Insertion Tool:** After removing the O'Ring to be replaced, carry the new O'Ring into the Cylinder and mount it until it is firmly seated in the hollow part.

SANITATION

This product should be correctly sanitized after each use. Proceed as follows: 1st step: Fully immerse the part in enzymatic detergent (diluted according to the manufacturer). 2nd step: Wash it in an ultrasonic cleaner for approximately 10 to 15 minutes. 3rd step: Rinse with distilled water in abundance, until completely remove the wastes from the solution. Nylon brushes use is allowed. 4th step: Dry it with a clean, dry cloth or with compressed air. 5th step: Perform a visual inspection, observing if there are failures in the cleaning process. If there are still wastes, the part should be once again immersed in detergent - 1st step - and, if necessary, the cleaning should be performed with the aid of a nylon brush. Repeat the rinsing and drying sequence. 6th step: Select the packing in accordance with the sterilization process. Preferably, use surgical grade paper self-sealing packing with laminated film. **ATTENTION:** Do not use antitrust products, avoid using enzymatic liquid in concentrations above 10% and dry parts that still have cleaning solution wastes, because these procedures cause oxidation. The use of steel brushes is also inadvisable.

PRESENTATION AND STERILIZATION

It is packaged in a blister-type package (film and surgical grade paper), and sold individually. It is not recommended to autoclave the product in its original packaging. It is reusable, provided in non-sterile form, and must therefore be sterilized before use. Sterilize the product the day before or on the day of the procedure. It is recommended, preferably, to follow the method of sterilization using a steam autoclave, and also the parameters and procedures established in standard BS EN ISO 17665-1:2006 (Sterilization of health care products. Moist heat. Requirements for the development, validation and routine control of a sterilization process for medical devices). Alternatively, you can proceed according to the instruction manual from the manufacturer of the autoclave. Recommended parameters: 121°C, 30 min, at 1 atm. Validity of the sterilization: 7 to 15 days, provided that the product is stored in a clean, dry environment away from sunlight.

PRECAUTIONS

- Do not use the product if the packing is damaged.
- The O'Ring Insertion/Extractor Tool tilt during torque application may result in it being damaged and/or fractured.
- Better results are obtained with the use of the Neodent products sequence. The use of instruments and/or Prosthetic Abutments of other manufacturers does not guarantee the perfect function of the Neodent Implants System and exempts any guarantee of the product.
- Improper planning can jeopardize the performance of the Implant/prosthesis set, causing failures to the system, such as loss or break of the Implant, loosening or fracture of the Prosthetic Screws.

ADVERSE EFFECTS

Adverse effects will only occur if the choice and the use of instrument are improper.

POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring after the surgery and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

DATE OF EXPIRATION

Written on the label.

LIFE CYCLE

This product should be disposed of after it loses its functionality.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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