ST.330.215

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Español  Protector de Pulido

English  Polishing Protector

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

DESCRIPTION

The Neodent Polishing Protector is manufactured in stainless steel. It has your fixation region configured in accordance to the adaptation geometry of the Prosthetic Component to be polished. It is a Component exclusively developed to prosthetic work.

APPLICATIONS

The Neodent Polishing Protector is a device that protects the Prosthetic Components during the polishing procedure. Therefore, it is intended for exclusively use of prosthetic laboratories.

CONTRAINDICATIONS

This product has no contraindications, providing its intended purposes and indications are correctly followed.

HANDLING

The Neodent Polishing Protector is placed into the Prosthetic Component and fixed with a Screw in the lower region. Its function is protect the adaptation region of the Prosthetic Component. After that, is made the polishing, wearing or finishing of the prosthetic work.

PRESENTATION AND STERILIZATION

It is supplied unitarily, conditioned in a blister type package (surgical grade paper and film). It is reusable and delivered non-sterile. Note: For reuse, clean the Neodent Polishing Protector with an air jet or wash with water.

PRECAUTIONS

- Do not use the product if the package has been violated.
- Better results are achieved with the utilization of the Neodent products sequence. The utilization of instrumentations and/or Prosthetic Components from other manufacturers does not assure the perfect function of the Neodent Implant System and voids any product guarantee.
- The improper planning may endanger the Implant/Prosthesis set performance resulting in faults in the system, such as Implant loss or fracture, loosening or fracturing of the Prosthetic Screws.

STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

DATE OF EXPIRATION

Written on the label.

LIFE CYCLE

This product should be discarded when it loses its functionality.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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