ST.330.220

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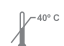
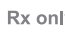
Español  Conexión Philips

English  Philips Connection

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Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Tamanho do produto / Product size / Tamaño del producto / Dimensioni del prodotto / Produktmaße / Taille du produit
	Código do produto / Product code / Código del producto / Codice del prodotto / Produktcode / Code du produit
	Número do lote / Batch Number / Número de Partida / Numero di lotto / Chargennummer / Numéro de lot
	Data de fabricação / Date of manufacture / Fecha de fabricación / Data di fabbricazione / Herstellungsdatum / Date de fabrication
	Prazo de validade / Shelf life / Plazo de validad / Durata di conservazione / Mindesthaltbarkeit / Date de péremption
	Quantidade / Quantity / Cantidad / Quantità / Qualität / Quantité
	Material de fabricação do produto / Material used / Material de fabricación del producto / Materiale di fabbricazione del prodotto / Herstellungsmaterial des Produkts / Matériau de base du produit
	Não estéril / Non-sterile / No estéril / Non sterile / Nicht steril / Non stérile

Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Consulte as instruções de utilização / Refer to instructions for use / Consulte las instrucciones de utilización / Consultare le istruzioni per l'uso / Siehe Bedienungsanleitung / Consulter les instructions d'utilisation
	Limite superior de temperatura / Upper limit of temperature / Limite superior de temperatura / Limite di temperatura massima / Maximaltemperatur / Limite de température supérieure
	Conservar seco / Keep dry / Consérvelo seco / Conservare in luogo asciutto / Trocken lagern / Garder au sec
	Manter afastado da luz solar / Keep protected from sunlight / Manténgalo lejos de la luz solar / Tenere al riparo dalla luce solare / Vor Sonneneinstrahlung schützen / Conserver à l'abri de la lumière solaire
	Não utilizar se a embalagem estiver danificada / Do not use if package is damaged / No lo utilice si el envoltorio está dañado / Non utilizzare se la confezione originale è danneggiata / Bei beschädigter Verpackung Produkt nicht verwenden / Ne pas utiliser si l'emballage a été abîmé

Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Marcação CE para comercialização na Comunidade Europeia / CE Mark for European Community market / Marca CE para comercialización en la Comunidad Europea / Marchio CE per la commercializzazione in Europa / CE Kennzeichen zum Vertrieb in der Europäischen Gemeinschaft / Marquage CE pour une commercialisation en Europe
	Notificação exigida pelo FDA para comercialização nos Estados Unidos / Notification required by FDA for United States market / Notificación exigida por el FDA para comercialización en los Estados Unidos / Notifica necessaria per la commercializzazione negli Stati Uniti / Vom FDA geforderte Kennzeichnung für den Vertrieb in den USA / Notification exigée par FDA pour une commercialisation aux États-Unis
	Representante na Comunidade Europeia / Representative in the European Community / Representante en la Comunidad Europea / Rappresentante nella Comunità europea / Vertreter in der Europäischen Gemeinschaft / Représentant dans la Communauté Européenne
	Fabricante / Manufacturer / Fabricante / Produttore / Hersteller / Fabricant

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

DESCRIPTION

The Philips Connection for Manual ScrewDriver Neodent has cylindrical shape and Philips fit at its tip. It has also an circular orifice, responsible for locking the internal part of the Bone Grafting Manual Driver. The Philips Connection for Contra-Angle Neodent has cylindrical shape, fit for Contra-Angle at one end and Philips fit in the another one. It is indicated for capture and hold the Graft Screw and fix it in the bone tissue, with the use of a micromotor and Contra-Angle. The Philips Connections Neodent are manufactured in surgical stainless steel, supplied non-sterile and can be reused in accordance with the guidelines and precautions contained in these usage instructions.

APPLICATIONS

The Philips Connection for Manual ScrewDriver Neodent is an intermediary between the Bone Grafting Manual Driver and the Graft Screw, indicated to capture and hold the Screw, fixing the bone block to the receptor region. The Philips Connection- Contra-Angle Neodent is an intermediary between the Contra-Angle and the Graft Screw, indicated to capture and hold the Screw, fixing the bone block to the receptor region.

CONTRAINDICATIONS

This product has no contraindications, providing its intended purposes and indications are correctly followed.

HANDLING

After the Connection fitting, press the Connection active tip over the Graft Screw. Install the Screw in the bone tissue. Remove the Connection. To remove the Screw, do the same procedure, but in the opposite direction.

SANITATION

The Philips Connections Neodent and all surgical instruments should be correctly cleaned after each use. Do not leave in contact with dampness for prolonged periods. Follow these steps for cleaning: 1st step: Disassemble the part (if applicable). 2nd step: Submerge the parts fully into enzymatic detergent (diluted according to the manufacturer). 3rd step: Wash in ultrasonic washer for approximately 10 to 15 minutes. 4th step: Rinse with plenty of distilled water until the residue of the solution has been completely removed. Always use nylon brushes. The use of steel brushes is not recommended. 5th step: Dry with a paper towel or compressed air. 6th step: Perform a visual inspection observing whether there are any faults in the cleaning process. If there are still residues, you must repeat the 2nd Step. 7th step: Select the packaging according to the sterilization process. Preferably use self-sealing package of medical grade paper with laminated film. ATTENTION: Do not use incrustation removal products, avoid using enzymatic liquid in concentrations above 10%, and drying parts that still contain residues from cleaning solutions, as these procedures favor oxidation. The use of steel brushes is also not recommended. Drying parts prior to storage and sterilization is extremely important, as buildup of humidity in them is detrimental and could cause spots of oxidation.

STERILIZATION

Sterilize the product on the day before or on the day of the procedure. We preferably recommend following the sterilization method by steam autoclaving and, yet, the parameters and procedures established in standard ISO 17665-1:2006 - Sterilization of health care products. Moist heat. Requirements for the development, validation and routine control of a sterilization process for medical devices. Suitable parameters: 121 °C, 30 min, at 1 atm. Alternatively, can proceed according to the instruction manual of the manufacturer of the autoclave. Sterilization validity: 7 to 15 days, provided that the product is conditioned in a clean, dry environment away from the sunlight.

PRESENTATION AND STERILIZATION

The Philips Connections Neodent are protected by a blister type packaging (film and medical grade paper) and supplied by unit. This product is reusable, supplied non-sterile and, therefore, should be sterilized before use.

PRECAUTIONS

- Do not use the product if the package has been violated.
- This product should be used in its sterile condition.
- Better results are achieved with the utilization of the Neodent products sequence. The utilization of instrumentations and/or Prosthetic Components from other manufacturers does not assure the perfect function of the Neodent Implant System and voids any product guarantee.
- The improper planning may endanger the Implant/Prosthesis set performance resulting in faults in the system, such as Implant loss or fracture, loosening or fracturing of the Prosthetic Screws.

ADVERSE EFFECTS

Adverse effects will only occur if the instruments are chosen incorrectly.

STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

DATE OF EXPIRATION

Written on the label.

LIFE CYCLE

This product should be disposed of when it loses its functionality.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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