ST.330.227

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This device is intended for the specialized procedure, which should be performed by professionals trained. For better results, use the product with the knowledge of the suitable techniques. Always accomplish them in suitable conditions, including surgical room.

DESCRIPTION

Neodent Ratchet Wrenches are produced with stainless steel, and all have a gear (1) at their end for the fitting of Torque Connections, and an internal shaft (2) that allows the rotation direction of the Ratchet (clockwise and anti-clockwise). The Ratchet Wrench with the torque option allows the tightening torque (clockwise direction) to be monitored, with the aid of a torque shaft. The direction of the arrow on the pin of the shift indicates the torque direction.

APPLICATIONS

The Neodent Ratchet Torque Wrench is a tool that supports insertion and torque control. It can be used in surgical and prosthetic procedures.

CONTRAINDICATIONS

The Neodent Torque Wrenches have no contraindications, providing their intended purposes and indications are correctly followed.

ASSEMBLY AND DISASSEMBLY

-Assembly: a) Fit the gear (1) into the Torque Wrench body (4). Make sure that the gear is inserted through the lower of the Torque Wrench body, as shown in figure (5). b) Insert the axis (2) and thread it until the perfect fitting in the Torque Wrench body.

-Disassembly: a) Remove the axis. b) Remove the gear.

ATTENTION: The figures above are for illustrative purposes only. They do not represent the real dimensions and characteristics of the product surface.

HANDLING

After fitting the Torque Connection in the Ratchet Driver gear, support the gear with one hand, and use your other hand to pull the Ratchet Driver. In the case of the Torque Wrench, pull the Torque Wrench by the rod (6) to monitor the torque in every movement of the Torque Wrench (7). The torque can be of 10, 20, 32, 45 and 60 N.cm. To reverse the rotation direction of the Torque Wrench, pull and rotate 180o the ratchet axis pin, leaving the arrow pointing toward the desired direction.

SANITATION

The Neodent Torque Wrenches should be correctly cleaned after use. To clean, follow these steps: 1st step: Always disassemble the parts (as described in the item Assembly and Disassembly). 2nd step: Submerge the parts fully into enzymatic detergent (diluted according to the manufacturer). 3rd step: Wash in ultrasonic washer for approximately 10 to 15 minutes. 4th step: Rinse with plenty of distilled water until the residue of the solution has been completely removed. Always use nylon brushes. The use of steel brushes is not recommended. 5th step: Dry with a paper towel or compressed air. 6th step: Perform a visual inspection observing whether there are any faults in the cleaning process. If there are still residues, you must repeat the 2nd Step. 7th step: Select the packaging according to the sterilization process. Preferably use self-sealing package of medical grade paper with laminated film. ATTENTION: Do not use incrustation removal products, avoid using enzymatic liquid in concentrations above 10%, and drying parts that still contain residues from cleaning solutions, as these procedures favor oxidation. The use of steel brushes is also not recommended. Drying parts prior to storage and sterilization is extremely important, as buildup of humidity in them is detrimental and could cause spots of oxidation.

FORMS OF PRESENTATION AND STERILIZATION

The Neodent Torque Wrenches are reusable, supplied non-sterile and therefore should be sterilized before use. Remove the part and sterilize the product on the day before or on the day of the procedure. We preferably recommend following the sterilization method by steam autoclaving and, yet, the parameters and procedures established in standard ISO 17665-1:2006 • Sterilization of health care products. Moist heat. Requirements for the development, validation and routine control of a sterilization

process for medical devices. Suitable parameters: 121 °C, 30 min, at 1 atm. Alternatively, can proceed according to the instruction manual of the manufacturer of the autoclave.

PRECAUTIONS

The use of excessive force may compromise the surgical act. When reversing the direction of the Ratchet Torque Wrench, the gear becomes loose and may fall off. Therefore, always perform this procedure outside the patient's mouth. Do not use the product if the packaging has been tampered with. Do not autoclave the Wrench in its original packaging. Always sanitize and autoclave the part when disassembled. Best results obtained with use of the sequence of Neodent products. The use of instruments and/or prosthetic components from other manufacturers does not ensure the perfect function of the Neodent Implant System and disclaims any warranty.

ADVERSE EFFECTS

Adverse effects will only occur if instruments are chosen incorrectly, which could damage the implant to be installed.

STORAGE CONDITIONS

This product should be stored in a clean and dry location, at a maximum temperature of 40 °C and protected from the direct sunlight radiation.

DISPOSAL OF MATERIALS

After use, all consumer materials used in a surgery for the installation of dental implants could offer risks to the health of those handling them. Prior to disposal in the environment, we recommend that users refer to and comply with current legislation.

SHELF LIFE

The shelf life of these products is indicated on the label.

LIFE CYCLE

The Neodent Ratchet Drivers should be disposed when they lose their functionality.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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