ST.330.228

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English  Drivers

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Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Tamanho do produto / Product size / Tamaño del producto / Dimensioni del prodotto / Produktmaße / Taille du produit
	Código do produto / Product code / Código del producto / Codice del prodotto / Produktcode / Code du produit
	Número do lote / Batch Number / Número de Partida / Numero di lotto / Chargennummer / Numéro de lot
	Data de fabricação / Date of manufacture / Fecha de fabricación / Data di fabbricazione / Herstellungsdatum / Date de fabrication
	Prazo de validade / Shelf life / Plazo de validad / Durata di conservazione / Mindesthaltbarkeit / Date de péremption
	Quantidade / Quantity / Cantidad / Quantità / Qualität / Quantité
	Material de fabricação do produto / Material used / Material de fabricación del producto / Materiale di fabbricazione del prodotto / Herstellungsmaterial des Produkts / Matériau de base du produit
	Não estéril / Non-sterile / No estéril / Non sterile / Nicht steril / Non stérile

Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Consulte as instruções de utilização / Refer to instructions for use / Consulte las instrucciones de utilización / Consultare le istruzioni per l'uso / Siehe Bedienungsanleitung / Consulter les instructions d'utilisation
	Limite superior de temperatura / Upper limit of temperature / Limite superior de temperatura / Limite di temperatura massima / Maximaltemperatur / Limite de température supérieure
	Conservar seco / Keep dry / Consérvelo seco / Conservare in luogo asciutto / Trocken lagern / Garder au sec
	Manter afastado da luz solar / Keep protected from sunlight / Manténgalo lejos de la luz solar / Tenere al riparo dalla luce solare / Vor Sonneneinstrahlung schützen / Conservar à l'abri de la lumière solaire
	Não utilizar se a embalagem estiver danificada / Do not use if package is damaged / No lo utilice si el envoltorio está dañado / Non utilizzare se la confezione originale è danneggiata / Bei beschädigter Verpackung Produkt nicht verwenden / Ne pas utiliser si l'emballage a été abimé

Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Marcação CE para comercialização na Comunidade Europeia / CE Mark for European Community market / Marca CE para comercialización en la Comunidad Europea / Marchio CE per la commercializzazione in Europa / CE Kennzeichen zum Vertrieb in der Europäischen Gemeinschaft / Marquage CE pour une commercialisation en Europe
	Notificação exigida pelo FDA para comercialização nos Estados Unidos / Notification required by FDA for United States market / Notificación exigida por el FDA para comercialización en los Estados Unidos / Notifica necessaria per la commercializzazione negli Stati Uniti / Vom FDA geforderte Kennzeichnung für den Vertrieb in den USA / Notification exigée par FDA pour une commercialisation aux États-Unis
	Representante na Comunidade Europeia / Representative in the European Community / Representante en la Comunidad Europea / Rappresentante nella Comunità europea / Vertreter in der Europäischen Gemeinschaft / Représentant dans la Communauté Européenne
	Fabricante / Manufacturer / Fabricante / Produttore / Hersteller / Fabricant

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

DESCRIPTION

- Manual Driver and Manual Screwdriver - Torque Connection: they have a cylindrical shape for square fit in the active end, to be used with the Drivers - Torque Connection.
- Torque Wrench Adaptor Connection - Contra-Angle: it has a cylindrical shape with a fit for Contra-Angle Connection in the active end and a square coupling in the other end, to be used with the Manual Driver and Manual Screwdriver - Torque Connection.
- Manual Screwdriver - Contra-Angle: it has a cylindrical shape with fit for Contra-Angle Connection in the active end.
- Impression Coping Driver: it has a cylindrical shape with conic fit in the active end for coupling of the Closed Tray Impression Copings.
- Handle Anchorage Implant Driver: it has a cylindrical shape with inner hexagon fit at the end for Self-Drilling Anchorage Implant.
- Zygomatic Installation Driver: it has an ergonomic grip handle and a square fit in the active end for coupling on the Zygomatic Implant Mount. They are made in stainless steel.

APPLICATIONS

They are intended for:

- Manual Driver - Torque Connection: manual installation of Implants in anterior regions.
- Manual Screwdriver - Torque Connection: helping the Torque Connection handling.
- Torque Wrench Adaptor Connection - Contra-Angle: it allows the use of Contra-Angle Connections in Manual Screwdriver - Torque Connections.
- Manual Screwdriver - Contra-Angle: it allows the use of Contra-Angle Connections in Manual Screwdriver.
- Impression Coping Driver: capture, transportation and handling of the Closed Tray Impression Copings.
- Handle Anchorage Implant Driver: manual installation of Self-Drilling Anchorage Implants.
- Zygomatic Installation Driver: manual installation of Zygomatic Implants.

CONTRAINDICATIONS

This product does not show contraindications, provided that it is properly used for the shown purposes

HANDLING

Manual Driver - Torque Connection: Fit the active end of the Manual Driver into the square end of the Torque Connection and install the Connection into the surgical cavity, turning it clockwise, manually. Manual Screwdriver - Torque Connection: Fit the active end of the Manual Screwdriver into the square end of the Torque Connection and install the Connection into the surgical cavity, turning it clockwise, manually. Torque Wrench Adaptor Connection -

Contra-Angle: fit the square end of the Adaptor into the square fit of the Manual Screwdriver - Torque Connection. Next, couple the selected Contra-Angle Connection in the Adapter. Manual Screwdriver - Contra-Angle: Couple the selected Contra-Angle Connection and proceed with the use in the surgical cavity, manually. Impression Coping Driver: Couple the selected Impression Coping and proceed with the insertion in the Implant, manually. Handle Anchorage Implant Driver: fit the Handle Anchorage Implant Driver hexagon in the hexagon part of the Self-Drilling Anchorage Implant and install it in the surgical cavity (turning it clockwise) by hand. Zygomatic Installation Driver: Fit the active end of the Driver in the square end of the Zygomatic Mount and proceed with the insertion of the Zygomatic Implant in the surgical cavity, turning it clockwise by hand. To remove it, use the same fitting system, turning anti-clockwise. **ATTENTION: PLEASE AVOID SIDE LOADS. THIS PRODUCT DOES NOT HAVE A TORQUE CONTROL. THE USE OF EXCESSIVE FORCE MAY DAMAGE THE PRODUCT.**

SANITATION

The surgical instrumentation should be properly sanitized after its utilization. Do not let the product contact moisture beyond the time required for the sanitation. For such, proceed as follows: Step 1: Submerge the parts fully into enzymatic detergent (diluted according to the manufacturer). Step 2: Wash in ultrasonic washer for approximately 10 to 15 minutes. Step 3: Rinse with plenty of distilled water until the solution wastes are fully removed. The utilization of nylon brushes is allowed. Step 4: Dry with a clean and dry cloth or with compressed air. Step 5: Perform a visual inspection, watching if there are faults in the cleaning process. If any wastes are still left, the part should be submerged again in detergent - step 1 - and if necessary, the cleaning should be made with the help of a nylon brush. Repeat the rinsing and drying sequence Step 6: If applicable, place the product in a suitable case duly sanitized and dried. Step 7 - Select the package according to the sterilization process. Use preferably a self-sealing package if surgical grade paper with laminated film. **ATTENTION: Do not use descaling products and avoid using enzymatic liquid in concentrations higher than 10% and dry the parts still containing cleaning solution wastes because such procedures favor the oxidation. Also, the use of steel brushes is not recommended.**

PRESENTATION AND STERILIZATION

This product is supplied unitarily and is conditioned in a plastic package. It is reusable, delivered non-sterile and it should, therefore, be sterilized before its use. Sterilize the product on the eve or in the procedure day. Preferably, it is recommended to follow the sterilization method by steam autoclaving and, yet, the parameters and procedures established in standard ISO 17665-1:2006 - Sterilization of health care products. Moist heat. Requirements

for the development, validation and routine control of a sterilization process for medical devices. Suitable parameters: 121°C, 30 min, at 1 atm. Alternatively, can proceed according to the instruction manual of the manufacturer of the autoclave. Sterilization validity: 7 to 15 days, provided that the product is conditioned in clean, dry and protected from the sunlight environment.

PRECAUTIONS

- Do not use the product if the package has been violated.
- This product should be used in its sterile condition.
- Avoid lateral loading.
- This product does not have torque control. The use of excessive force may damage the product.
- Better results are achieved with the utilization of the Neodent products sequence. The utilization of instrumentations and/or Prosthetic Components from other manufacturers does not assure the perfect function of the Neodent Implant System and voids any product guarantee.
- The improper planning may endanger the Implant/Prosthesis set performance resulting in faults in the system, such as implant loss or fracture, loosening or fracturing of the Prosthetic Screws.

ADVERSE EFFECTS

Adverse effects will only occur if the instrumentation selection and use are improper.

POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring after the surgery and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

DATE OF EXPIRATION

Written on the label.

LIFE CYCLE

This product should be disposed of when it loses its functionality.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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