# ST.330.264

Português 

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

## DESCRIPTION

The Bone Grafting / Anchorage Drill is manufactured in surgical stainless steel. In its active end there are cylindrical profile and high cutting power. In the other end, there is available in two presentations: • fitting for Contra-Angle in accordance to standard ISO 1797-1 - Dental rotary instruments - Shank, and • fitting for straight piece. Laser markings determine the drilling depth in accordance to the surgical planning.

## APPLICATIONS

This product is a surgical instrument used for drilling the bone tissue, with the purpose of preparation: • For the insertion of Self-Drilling Anchorage Implants, when it is not possible to use the self-drilling function, especially in bones types I and II. • For drilling the intraoral bone grafting receiving region and drilling the bone block (removed from the donor region), when it is not possible to use the self-drilling function of the Screws for grafting, especially in bones types I and II.

## CONTRAINDICATIONS

This product does not present contraindications, once they are properly used for the indicated purposes.

## HANDLING

Fit the Drill in the straight part or in the Contra-Angle. Place it in the specific rotation, according to instructions for use of the Self-Drilling Anchorage Implant or of the Screw for Bone Grafting chosen. Perform the drilling with complete insertion and removal movements of the Drill. During drilling, the pressure should not be excessive and, abundant irrigation, manual or combined, using the handpiece pump is necessary. Note 1: In order to guarantee the stability of the Screw for grafting or of the Self-Drilling Anchorage Implant, respect the drilling diameter of each case, according to instructions of use of the Implant or Screw chosen. Note 2: Do not interrupt the handpiece rotation with the Drill within the surgical cavity, because this can make difficult the removal or cause the Drill break. Note 3: Due to the reduced diameter, do not lean the Drill during drilling.

## SANITATION

The surgical instruments should be correctly sanitized after each use. Do not leave the product in contact with humidity more than the time necessary for the sanitation. Proceed as follows: 1<sup>st</sup> step: Fully immerse the part in enzymatic detergent (diluted according to the manufacturer) 2<sup>nd</sup> step: Wash in ultrasonic cleaner for approximately 10 to 15 minutes. 3<sup>rd</sup> step: Rinse with distilled water in abundance, until completely remove the wastes from the solution. Nylon brushes use is recommended. 4<sup>th</sup> step: Dry with a clean and dry cloth or with compressed air. 5<sup>th</sup> step: Perform a visual inspection, observing if there are failures in the cleaning process. If there are still wastes, the part should be once again immersed in detergent - 1<sup>st</sup> step - and, if necessary, the cleaning should be performed with the aid of a nylon brush. Repeat the rinsing and drying sequence. 6<sup>th</sup> step: If applicable, place the product in an appropriate case duly sanitized and dry. 7<sup>th</sup> step: Select the packing in accordance to the sterilization process. Use preferably surgical grade paper self-sealing packing with laminated film.

## PRESENTATION AND STERILIZATION

This product is unitarily provided, conditioned in blister (surgical grade film and paper type) packing. It is reusable, delivered non-sterile and should, therefore, be sterilized before use. Sterilize the product on the day before or on the day of the procedure. We preferably recommend following the sterilization method by steam autoclaving and, yet, the parameters and procedures established in standard ISO 17665-1:2006 - Sterilization of health care products. Moist heat. Requirements for the development, validation and routine control of a sterilization process for medical devices. Suitable parameters: 121°C, 30 min, at 1 atm. Alternatively, can proceed according to the instruction manual of the manufacturer of the autoclave. Sterilization validity: 7 to 15 days, provided that the product is conditioned in clean, dry and protected from the sunlight environment.

## PRECAUTIONS

• Do not use the product if the packing is damaged. • This product should be sterile. • The non-replacement of the Drills, as recommended by the manufacturer, can generate improper bone heating, compromising the osseointegration success. • Do not regrind. • Do not use solutions with ammonia, acids or hydrogen peroxide, because they can damage the drill. • Do not use antitrust products, avoid using enzymatic liquid in concentrations above 10% and dry parts that still have cleaning solution wastes, because these procedures cause oxidation. • The use of

steel brushes is inadvisable. We recommend the use of nylon brushes. • The improper planning can jeopardize the performance of the Implant/ Prosthesis set causing failures to the system, such as loss or break of the Implant, loosening or fracture of the Prosthetic Screws. • Better results are obtained with the use of the Neodent products sequence. • The use of instruments and/ or Prosthetic Abutments of other manufacturers do not guarantee the perfect function of the Neodent Implants System and exempts any guarantee of the product.

## ADVERSE EFFECTS

Adverse effects will only occur if the choice and the use of instrument are improper.

## POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring after the surgery and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

## STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

## DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

## DATE OF EXPIRATION

Written on the label.

## LIFE CYCLE

This product has a life cycle of up to 20 drillings for bone qualities I and II, and up to 30 drillings for bone qualities III and IV, since its use conditions are respected, including adequate irrigation and rotation. Apart of the life cycle period mentioned, the professional should check the sharpening conditions of the drill. Drills with improper cutting conditions should be disposed. Note: Classification of the bone quality according to Lekholm and Zarb (1985).

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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