ST.330.265

Português  Posicionador O'ring

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English  O'ring Positioner

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

DESCRIPTION

This product is manufactured in polymer and is available in three colors and angles. White • 0°; Green - 7°; Red -14°. It is presented in a cylindrical shape with a socket for coupling in the Ball Attachment.

APPLICATIONS

It is indicated to determine the proper distance between the Ball Attachment and the O'ring, and when required, to correct the Overdenture prosthesis insertion axle (parallelism) during the O'ring capture procedures. The Positioner selection depends on the tilt between each Implant.

CONTRAINDICATIONS

This product does not present contraindications, provided that it is used properly for the assigned purposes.

HANDLING

Couple the Positioner in the Ball Attachment, using it as a spacer during the O'rings capture. At the end of the capture procedure, remove it from the oral cavity.

PRESENTATION AND STERILIZATION

It is conditioned in blister type package (surgical grade paper or film). It is delivered non-sterile and is of single use.

PRECAUTIONS

- As to the systemic aspect, consider the patient's general health condition according to the applicable literature. As to the local aspect, observe the condition of the intraoral tissues.
- This product is intended for single use.
- Reuse of this product may cause:
 - adverse biological effects of residual products, microorganisms and / or substances resulting from previous uses and / or reprocessing;
 - changes in physical, mechanical and chemical properties of products, macro and micro structural, that can put in risk the desired functionality. The reuse of this product does not guarantee its safety and efficacy and disclaims any warranty of products.
 - Do not use the product if the package is violated.
 - Better results are achieved with the utilization of Neodent products. The utilization of instruments and/or Implants and/or Prosthetic Abutments of other systems does not assure the perfect function and voids any product guarantee
 - Make sure that the parts are not swallowed or inhaled by the person who is receiving the product (patient).
 - The improper planning may endanger the Implant/Prosthesis set performance resulting in system faults, such as implant loss or fracturing, loosening or fracturing of the prosthetic screws.

ADVERSE EFFECTS

No adverse effects will occur, provided that the indications are complied with.

POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring after the surgery and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

DATE OF EXPIRATION

Written on the label.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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