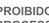
# ST.330.270

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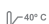
Español  O'Ring

English  O'Ring

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Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Tamanho do produto / Product size / Tamaño del producto / Dimensioni del prodotto / Produktmaße / Taille du produit
	Código do produto / Product code / Código del producto / Codice del prodotto / Produktcode / Code du produit
	Número do lote / Batch Number / Número de Partida / Numero di lotto / Chargennummer / Numéro de lot
	Data de fabricação / Date of manufacture / Fecha de fabricación / Data di fabbricazione / Herstellungsdatum / Date de fabrication
	Prazo de validade / Shelf life / Plazo de validad / Durata di conservazione / Mindesthaltbarkeit / Date de péremption
	Quantidade / Quantity / Cantidad / Quantità / Qualität / Quantité
	Material de fabricação do produto / Material used / Material de fabricación del producto / Materiale di fabbricazione del prodotto / Herstellungsmaterial des Produkts / Matériau de base du produit
	Requisito Brasileiro / Brazilian Requirement / Requisito Brasileiro / Requisito Brasiliano / Brasilianische Anforderungen / Exigences brésiliennes
	Não estéril / Non-sterile / No estéril / Non sterile / Nicht steril / Non stérile

Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Não reutilizar / Do not reuse / No lo reutilice / Non riutilizzare / Nicht wiederverwendbar / Ne pas réutiliser / Nicht wiederverwendbar / Ne pas réutiliser
	Consulte as instruções de utilização / Refer to instructions for use / Consulte las instrucciones de utilización / Consultare le istruzioni per l'uso / Siehe Bedienungsanleitung / Consulter les instructions d'utilisation
	Limite superior de temperatura / Upper limit of temperature / Limite superior de temperatura / Limite di temperatura massima / Maximaltemperatur / Limite de température supérieure
	Conservar seco / Keep dry / Consérvelo seco / Conservare in luogo asciutto / Troocken lagern / Garder au sec
	Mantener afastado da luz solar / Keep protected from sunlight / Manténgalo lejos de la luz solar / Tenere al riparo dalla luce solare / Vor Sonneneinstrahlung schützen / Conserver à l'abri de la lumière solaire
	Não utilizar se a embalagem estiver danificada / Do not use if package is damaged / No lo utilice si el envoltorio está dañado / Non utilizzare se la confezione originale è danneggiata / Bei beschädigter Verpackung Produkt nicht verwenden / Ne pas utiliser si l'emballage a été abîmé

Simbologia Symbology Simbología Simbologia Symbolik Symboles	Descrição / Description / Descripción / Descrizione / Beschreibung / Description
	Marcação CE para comercialização na Comunidade Europeia / CE Mark for European Community market / Marca CE para comercialización en la Comunidad Europea / Marchio CE per la commercializzazione in Europa / CE Kennzeichen zum Vertrieb in der Europäischen Gemeinschaft / Marquage CE pour une commercialisation en Europe
	Notificação exigida pelo FDA para comercialização nos Estados Unidos / Notification required by FDA for United States market / Notificación exigida por el FDA para comercialización en los Estados Unidos / Notifica necessaria per la commercializzazione negli Stati Uniti / Vom FDA geforderte Kennzeichnung für den Vertrieb in den USA / Notification exigée par FDA pour une commercialisation aux États-Unis
	Representante na Comunidade Europeia / Representative in the European Community / Representante en la Comunidad Europea / Rappresentante nella Comunità europea / Vertreter in der Europäischen Gemeinschaft / Représentant dans la Communauté Européenne
	Fabricante / Manufacturer / Fabricante / Produttore / Hersteller / Fabricant

102.075, 102.108, 102.113, 102.115

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This device is intended for a specialized procedure, which should be performed by professionals trained in Prosthesis. Polymer devices for dental use in prostheses over Neodent Implants are used during the preparation of supported-implanted prostheses, planned according to each case, aiming for function and aesthetics. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions.

## DESCRIPTION

The O-ring is a prosthetic component made of polymer. It has the function of retaining the prosthesis (overdenture), being positioned between the Attachment and the Cylinder. The O-ring for Mini Ball Attachment is available in the pink and white options and the O-Ring for Equator Attachment is available in the pink and violet options. The colors represent the different degrees of retention, according to the table below:

Color of the O-Ring	Retention
Pink	Soft
White	Average
Violet	Rigid

## APPLICATIONS

It is indicated as set of retention of the prosthesis (overdenture) on the Attachment.

## CONTRAINDICATIONS

The system of Neodent overdenture over attachment is contraindicated for cases where the angulation between the implants or roots exceeds 30° for the Equator Attachment, and 28° for the Mini Ball Attachment.

## HANDLING

### CAPTURE OF THE O-RING IN THE MOUTH WITH CYLINDER

For use of the Mini Ball Attachment, fit the appropriate positioner over the Attachment to correct the parallel alignment between the dental implants or roots and determine the correct distance between the O-ring and Attachment. For use of the Equator Attachment, it is not necessary to use positioners.

Position the protector disc over the positioner, for the Mini Ball Attachment, or over the Equator Attachment. Fit the O-ring with Cylinder over the Attachment chosen. Perform a wear (clearance) in the region of the prosthesis that will receive the O-ring with cylinder. Do a test of the prosthesis to verify its adaptation to the assembly. Capture, in the mount, the O-ring with Cylinder, one at a time, with acrylic resin, keeping the prosthesis in occlusion until the resin completely cures. Remove the prosthesis, the

protector disc, and the positioner and perform the finishing on the prosthesis. The Attachment will remain in the mouth and the cylinder with the O-ring will remain on the prosthesis.

### CHANGING THE O-RING

Remove the prosthesis from the mouth. With the aid of the O-ring Remover, completely remove the O-ring from the cylinder. Sanitize the prosthesis before installing the new O-ring. Install the new O-ring with the aid of the O-ring Mounter on the cylinder, pressing it until it fits properly.

## PRESENTATION AND STERILIZATION

This product is indicated for single use and supplied non-sterile, packed individually into a package that offers dual protection.

## PRECAUTIONS

- It is recommended to use the Protection Disc.
- It is recommended to use the O-ring Positioner between the Mini Ball Attachment and the O-ring, to correct the parallel alignment between the dental implants or roots and determine the correct distance between O-ring and Attachment.
- The incorrect choice or absence of the O-ring Positioner may be detrimental to the functioning of the prosthesis and cause premature wear of the O-ring.
- During the finishing of the prosthesis, care must be taken not to damage the O-ring.
- This device should be replaced regularly, every 8 months or when it stops functioning. Instruct the patient to come to the clinic regularly to have the retention of the prosthesis checked.
- This product should not be sterilized.
- Observe the conditions of the intraoral tissues via clinical exams, X-rays, and/or tomography. Failure to perform the pre-surgical assessment may compromise the success of the procedure.
- Inadequate planning may compromise the performance of the implant assembly or root/prosthesis, resulting in failures of the system, such as loss or fracture of the implant or root, loosening or fracturing of the prosthetic components.
- This product is of single use and cannot be reused.
- Reuse of this product may cause:
  - adverse biological effects of residual products, microorganisms and / or substances resulting from previous uses and / or reprocessing;
  - changes in physical, mechanical and chemical properties of products, macro and micro structural, that can put in risk the desired functionality. The reuse of this product does not guarantee its safety and efficacy and disclaims any warranty of products.
- Do not use the product if the packaging is broken.
- Do not use the product with the validity expired.
- Before each procedure, make sure the pieces are properly seated.
- Ensure that the parts are not swallowed or aspirated by the patient.
- Some of the possible causes of unsuccessful bone integration and loss of the prosthesis during

the treatment are the following: unsuitable osteotomy, infections, deficient oral hygiene, occlusal trauma, systemic problems or diseases, low remaining bone quantity or quality, lack or failure of irrigation, use of instruments which are not specific and/or without cutting power and lack of specific training.

- Be sure you have all the instruments needed to perform the procedure according to the planning.
- Before each procedure check the conditions of the Neodent instruments, always respecting their useful life. Replace the instruments if there is damage, markings deleted, sharpening jeopardized, deformation and wear.
- Always use the Neodent product sequence. The use of prosthetic components and/or instruments of other manufacturers does not ensure the perfect function of the Neodent Implant System and exempts any product warranty.
- It is the professional's responsibility to use the Neodent products according to the instructions for use, and to determine whether it suits the individual situation of each patient.

## ADVERSE EFFECTS

No adverse effects will occur, provided that the indications are respected.

## POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

## STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

## DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the procedure may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

## DATE OF EXPIRATION

Written on the label.

## LIFE CYCLE

This product has a life span of 8 months.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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