# ST.330.273

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This device is intended for a specialized procedure, which should be performed by professionals qualified in Dental Implants. For optimum results, use the product knowing the appropriate techniques. Always apply them under appropriate conditions, also in an operating room atmosphere.

### DESCRIPTION

The Implant Remover is a surgical instrument manufactured in stainless steel. It has a square fitting and O-ring for coupling to the Torque Wrench. At its active end, its geometry is appropriate for coupling in the inner portion of the Implant to be removed.

### APPLICATIONS

Is indicated for remove Implants.

### CONTRAINDICATIONS

This product does not show contraindications, provided that it be properly used for the shown purposes.

### HANDLING

Couple the Remover in the Torque Wrench, place the rotation direction of the Torque Wrench counterclockwise and exerting digital pressure on the Torque Wrench in the long axis of the Implant begins to remove it handling the Torque Wrench.

### PRESENTATION AND STERILIZATION

It is conditioned in blister (surgical grade film and paper type) packing and provided unitarily. It is single use and delivered non-sterile. Should,

therefore, be sterilized before use, when used in clinical environments. Sterilize the product on the day before or on the day of the procedure. We preferably recommend following the sterilization method by steam autoclaving and, yet, the parameters and procedures established in standard BS EN ISO 17665-1:2006 (Sterilization of health care products. Moist heat. Requirements for the development, validation and routine control of a sterilization process for medical devices). Suitable parameters: 121°C, 30 min, at 1 atm. Alternatively, can proceed according to the instruction manual of the manufacturer of the autoclave. Sterilization validity: 7 to 15 days, provided the product is conditioned in clean, dry and protected from sunlight environment.

### PRECAUTIONS

- Do not use the product if the package is violated.
- In a clinical environment, this product should be used only after being sterilized.
- It is recommended to keep the Implant Remover axial to the long axis of the Implant to be removed;
- The inclination of Implant Remover, during the torque application may result in damage or fracture.
- Better results are achieved with the utilization of Neodent products. The utilization of instruments and/or Implants and/or Prosthetic Abutments of other systems does not assure the perfect function and voids any product guarantee.
- The improper planning may endanger the Implant/Prosthesis set performance resulting in system faults, such as Implant loss or fracturing, loosening or fracturing of the Prosthetic Screws.

### ADVERSE EFFECTS

Adverse effects will only occur if the choice and the use of instrument are improper.

### POST-OPERATIVE PRECAUTIONS AND MAINTENANCE

Instruct the patient as to the need of a professional medical monitoring after the surgery and to obey the guidelines regarding the precautions, hygiene and prescription of drugs. These guidelines are the responsibility of the professional in charge.

### STORAGE CONDITIONS

This product should be stored in a clean and dry location, in a maximum temperature of 40°C and protected from direct sunlight.

### DISPOSAL OF MATERIAL

Every product and consumable used during the surgery for the installation of dental implants may endanger the health of those who handle them after use. Before discarding them into the environment, it is recommended to take a look at the current legislation and adhere to it.

### DATE OF EXPIRATION

Written on the label.

Federal (USA) law restricts this device to sale by or on the order of a licensed dentist or physician.

Not all products are available in all countries. Please, contact the authorized distributor.

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